

IMPORTANT TEST KIT INFORMATION

PLEASE READ CAREFULLY FOR VALID TESTING

- The Abbott BinaxNOW Home Test is for at-home use and is packaged with a telehealth session to oversee test administration and result reporting.
- THIS IS NOT A SELF PROCTORED TEST.
- The test does require a smartphone or laptop/desktop computer with a web cam and internet connection to set up account and administer the test.
- You will need:
 - A Windows or MAC/Apple laptop with a webcam (or a desktop computer with an adjustable/moveable webcam), microphone and speaker, OR
 - An Android or Apple phone/tablet
 - An email address (for ohio.emed test results)
 - An Internet connection
- Please note: At this time, a testing session can be completed end to end using a computer with a webcam, a cell phone or other mobile device, or a combination of the two.
- You will need to create an account using the NAVICA smartphone/tablet app or at www.mynavica.abbott with a laptop or desktop computer that has an adjustable or moveable webcam, a speaker and a microphone.
- You will go to ohio.emed.com to start a testing session once you have your NAVICA account set up. You will need to your NAVICA information for your session. A trained telehealth professional will guide you through the testing.
- **DO NOT** scan the test kit in the NAVICA app.
- **DO NOT** open the test kit prior to creating your NAVICA account and starting the test session through ohio.emed.com or the test will be void.
- You must wait for instruction of kit use during your testing session.
- **IMPORTANT STEPS:**
 - Create an account with the NAVICA app from Abbott for smartphones/tablets downloaded or laptop/desktop computer by visiting www.mynavica.abbott
 - Once a NAVICA account is set up, then go to - ohio.emed.com to start a testing session. A testing session is only initiated by going to ohio.emed.com and selecting the “Start Testing” button.
 - You will follow closely the instruction given by the test proctor through ohio.emed.com. Do not open or scan anything on the test kit or in the test kit until you are given specific instruction from your proctor.
 - The [ohio.emed](http://ohio.emed.com) proctor will be scanning information inside your test kit.
 - Test results will be reported to the NAVICA app displaying the name of the person who took the test, date test was taken, and test result.
 - If you do not have a smartphone/tablet, a record of test results can be obtained by logging into your NAVICA account at www.mynavica.abbott.